
HOW TO FIT IN 4 SIMPLE STEPS:

WITH THE

SMARTELOCK® PLUS



ProKnee's proprietary patented quick release buckle

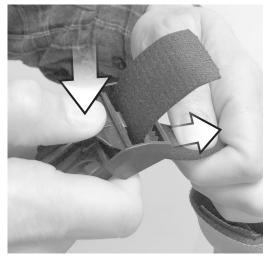

1. PUSH TAB TO LOOSEN STRAP





2. CLICK IN & PULL STRAP TO TIGHTEN & ACHIEVE DERSIRED FIT



3. THREAD STRAP THROUGH SLOT WITH TEETH



4. CUT EXCESS STRAP LENGTH
& LEAVE 3"
TIP: SINGE CUT END WITH

LIGHTER TO PREVENT FRAYING